

	Technical Information	730-188-EN		V01
	Weak printing of hand labelling guns	Created	21.03.2022	UK
		Changed		
		Checked	22.03.2022	UK
		Released	22.03.2022	UK
File no.: 5.1				

If the labels coming out of a labelling gun are not printed properly, the problem could come from three different reasons:

1. The printing bands may be uneven. This problem will happen only during the first use, not later with individual letters or numbers, not for several numbers or letters together or after several years of operation the printing bands may be worn out.
2. The printing force is too weak. There could be two reasons:
 - a. The pressing of the gun is too weak.
 - b. The gun has been used for a long time. We would recommend first to totally clean the gun with benzine or alcohol to make sure that the springs of the printing head are working well.
3. The ink roll is dry and should be exchanged. It could be possible that the ink is drying out when the roll is too old.

If the problem remains, don't hesitate to contact our application laboratory.