

<i>gke</i> – Technical Information	TI 730-141-EN	
Use of PCD-systems	Version 01	
	Created by	18.07.2016 UK
	Changed by	
	Checked by	26.08.2016 UK
	Released by	26.08.2016 UK
Page 1 of 1		

The different PCD-systems differ in penetration difficulty of the sterilant into the position of the indicator inside the PCD.

They are used:

1. As a type test according to the sterilizer standard (EN 285, 13060, 1422, 14180) to check if the sterilizer meets the sterilant penetration specifications used during IQ and OQ.
2. To assure correct sterilizer function after sterilizer start-up PCD systems according those standards shall be used (i.e. BD-Test - EN 285; Helix-Test – EN 867-5), called Bowie-Dick-Test.
3. For routine monitoring:
 - after validation of a process to assure sterility of a defined load.
 - The PCD shall simulate the worst case penetration load configuration.

Which PCD to use depends on the individual process and load configuration to monitor a sterilization process.

The construction of the ***gke*** Compact-PCDs is a patented design using 3 different volumes connected in series. Lengths and diameters do not provide any information on the real penetration characteristics of the PCD.

The ***gke*** application laboratory will support you to select the correct PCD-system.